


# gemini II treatment table

#6004

GEMINI II

---

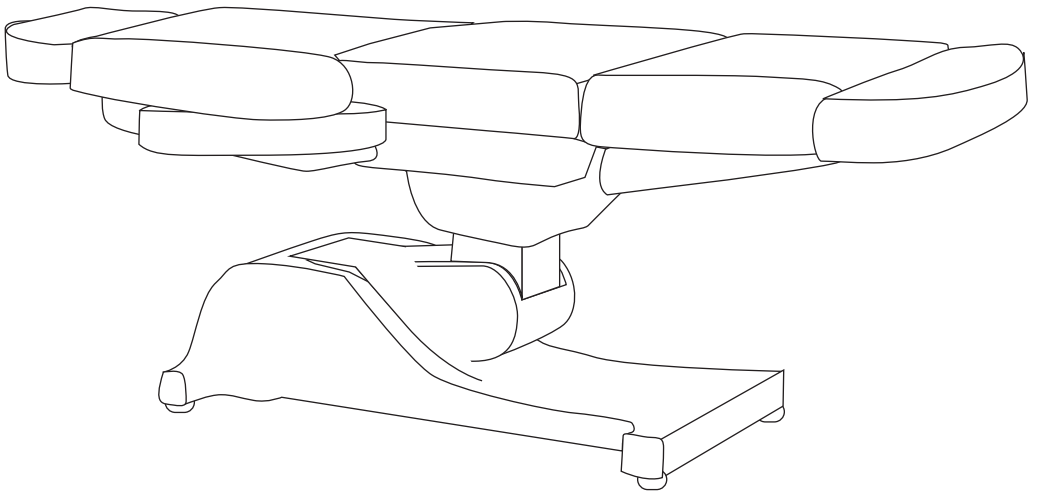



## IMPORTANT PLEASE READ THESE SAFETY INSTRUCTIONS

Listed below are essential safety precautions.

### GENERAL SAFETY

1. Read carefully and keep these instructions in a safe place.
2. Use this product only for its intended function and do not use attachments not recommended by the manufacturer.
3. Do not use outdoors.
4. Do not try to move the table when in use.

### ELECTRICAL APPLIANCES

1. All GPO's must be installed by a qualified Electrician in accordance with local electrical standards.
2. DO NOT allow children to operate the electric motors in this product.
3. Turn the power off and remove the plug when not in use.
4. Turn the power off and remove the plug when cleaning.
5. Do not let the power cord get damaged in any way.
6. Keep the power cord away from heat sources.
7. Do not use an extension cord unless the cord complies with local electrical safety standards.
8. Periodically inspect the power cord for signs of damage.
9. Do not operate any electrical item with a damaged power cord.



### IMPORTANT

KEEP FINGERS AND HANDS AWAY FROM TABLE JOINTS UNDER THE TABLE



### IMPORTANT

DO NOT ALLOW CHILDREN AROUND THE TREATMENT TABLE WHEN IN USE



### IMPORTANT

DO NOT ALLOW CHILDREN TO OPERATE THE ELECTRIC MOTORS THAT MOVE THE TABLE



### IMPORTANT

DO NOT USE AROUND WATER OR IN A WET ENVIRONMENT



### IMPORTANT

WHEN REPLACING ANY PARTS OR ASSEMBLING THE TABLE, YOU MUST ENSURE THE TABLE IS RESTING ON ITS SIDE TO SAFEGUARD FROM CRUSH INJURY

## GET STARTED

### GEMINI II

Beauty bed with electric adjustment

Width: 650mm (25.6") with arms 920mm (36.2")

Length: 1300mm-1810mm (51.2"-71.25")


Height Range: 620mm-1000mm (24.4"-39.4")

- Electric height, back, tilt & leg
- Hand held remote control




### OPTIONAL UPGRADE

**BATTERY** for back up power supply

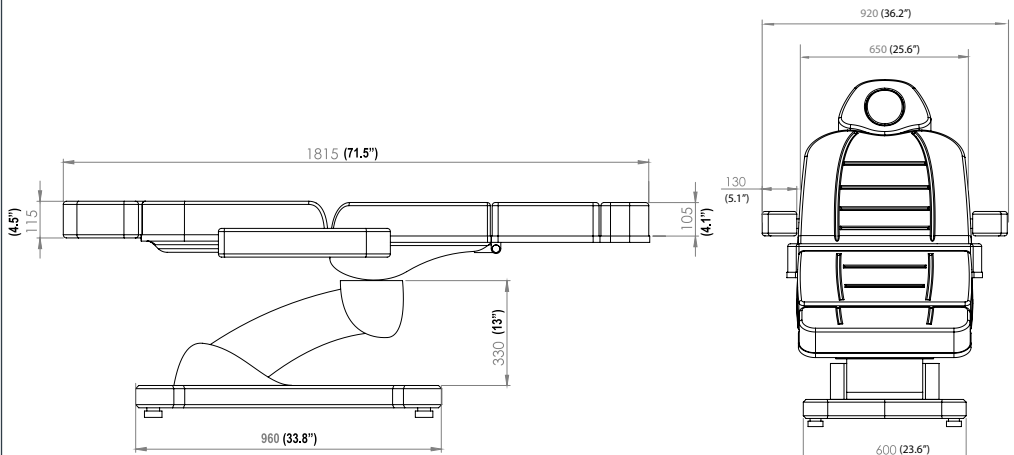

All measurements are in mm (inches).

## PARTS & ASSEMBLY

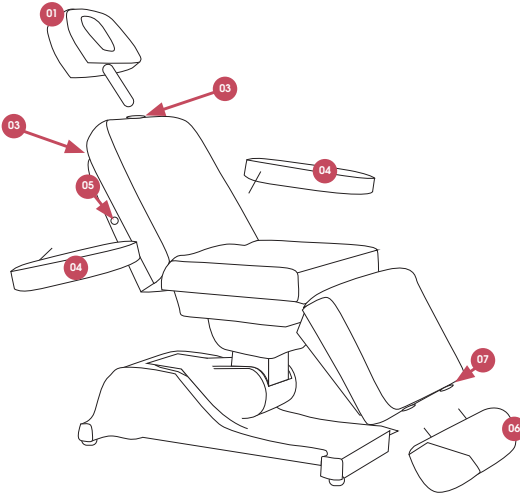


### PARTS

1. Removable headrest
2. Headrest Slot
3. Nut (at the rear of the chair)
4. Removable armrest
5. Armrest slot
6. Removable footrest
7. Footrest slot

### ASSEMBLY

1. Slot the headrest pole (02) into the slot provided (03) on the top of the chair. Tighten nut at the rear of the chair so headrest is secure.
2. Slot the footrest poles x2 (06) into the slots (07) provided on the end of the chair.
3. Guide each armrest (04) with the pole/bolt into each slot (05) on the left and right of the chair. If the armrest does not lock flat in position, please swap arms over.


## OPERATION

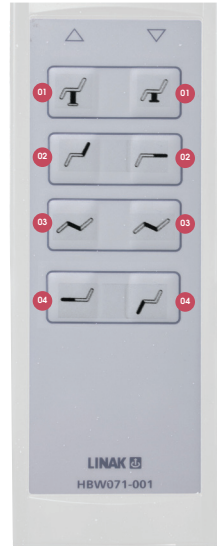

### HAND REMOTE CONTROL


1. Table up/down
2. Back Rest up/down
3. Tilt forward/back
4. Leg rest up/down

## FUNCTIONS

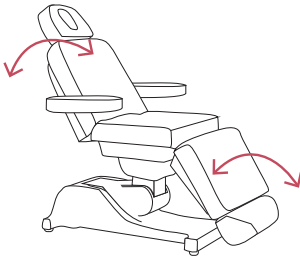


### BACK & LEG ADJUSTMENT

Using the hand remote control, adjust the back and leg adjustment by going up or down.

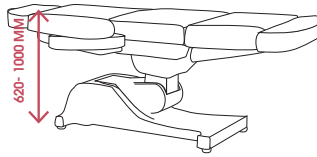

### HEIGHT ADJUSTMENT

Using the hand remote control, adjust the table height by going up or down.  
Minimum Height: 620mm  
Maximum height: 1000mm

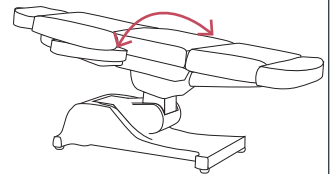

### TILT ADJUSTMENT

Using the hand remote control, adjust the tilt of the bed by going forward and back.

## MAINTENANCE

A periodic inspection of all components and fasteners should be made to ensure that your Comfortel Product is structurally intact and functioning properly.

Damaged and broken parts should be replaced and loose fasteners tightened.

All maintenance to be carried out by a suitably qualified tradesperson.

**⚠ IMPORTANT**  
WHEN REPLACING ANY PARTS OR ASSEMBLING THE TABLE, YOU MUST ENSURE THE TABLE IS RESTING ON IT'S SIDE TO SAFEGUARD FROM CRUSH INJURY

## CLEANING

### UPHOLSTERY

All upholstery needs some care; good housekeeping will help to prolong the life of any upholstered products. Avoid extreme temperatures. Extreme cold can make your vinyl prone to cracking, while extreme heat can cause permanent stretching. Avoid exposing your products to direct sunlight for prolonged periods of time.

### ALWAYS FOLLOW THE CLEANING INSTRUCTIONS

- Wipe with a clean colourfast cloth dampened with mild alkaline free soap and luke warm water. Do not wet excessively.
- Remove all traces of any cleaning product by wiping with fresh water and a clean cloth.
- Dry with a soft, clean cloth.
- Stubborn grime may require gentle scrubbing with a soft brush.

### Do not use:

- hot or boiling water
- hydrocarbons including alcohol
- acetone
- furniture polishes
- bleach
- concentrated detergents
- abrasive powder or paste cleaners
- steel wool
- silicon or petroleum-based cleaners
- solvent based or abrasive cleaning agents

# comfortel

[help.comfortel.com.au](http://help.comfortel.com.au)

### PRODUCT CODE

#6004 GEMINI II TREATMENT TABLE  
1 X 10AMP GPO REQUIRED  
POWER: 240V, 50-60HZ, MAX 2.4A  
MAX LOAD: 200KG (440lbs)

#6004-US  
POWER: 110V, 50-60HZ, MAX 2.4A  
MAX LOAD: 200KG (440lbs)

Revision Date: V3.1.09.2020

©Comfortel Furniture. All rights are reserved. No part of this document may be photocopied, reproduced or translated to another language without written consent of Comfortel Furniture Pty Ltd.

The information contained in this document is subject to change without notice and should not be construed as a commitment by Comfortel Furniture Pty Ltd.

Comfortel Furniture Pty Ltd. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Comfortel Furniture Pty Ltd. shall not be liable for incidental or consequential damages in connection with or arising out of the use of this document and the program material.

### PLEASE RETAIN THIS INFORMATION FOR FUTURE REFERENCE

For more information about our products and services, contact us at [comfortel.com.au](http://comfortel.com.au)

### WARRANTY PERIOD

TERMS AND CONDITIONS. For more information about our warranty, please refer to our website.

PURCHASE DATE

INVOICE NUMBER

NOTE YOUR PURCHASE DATE & INVOICE NUMBER FOR FUTURE REFERENCE