


vista

#6012.01 VISTA TREATMENT TABLE

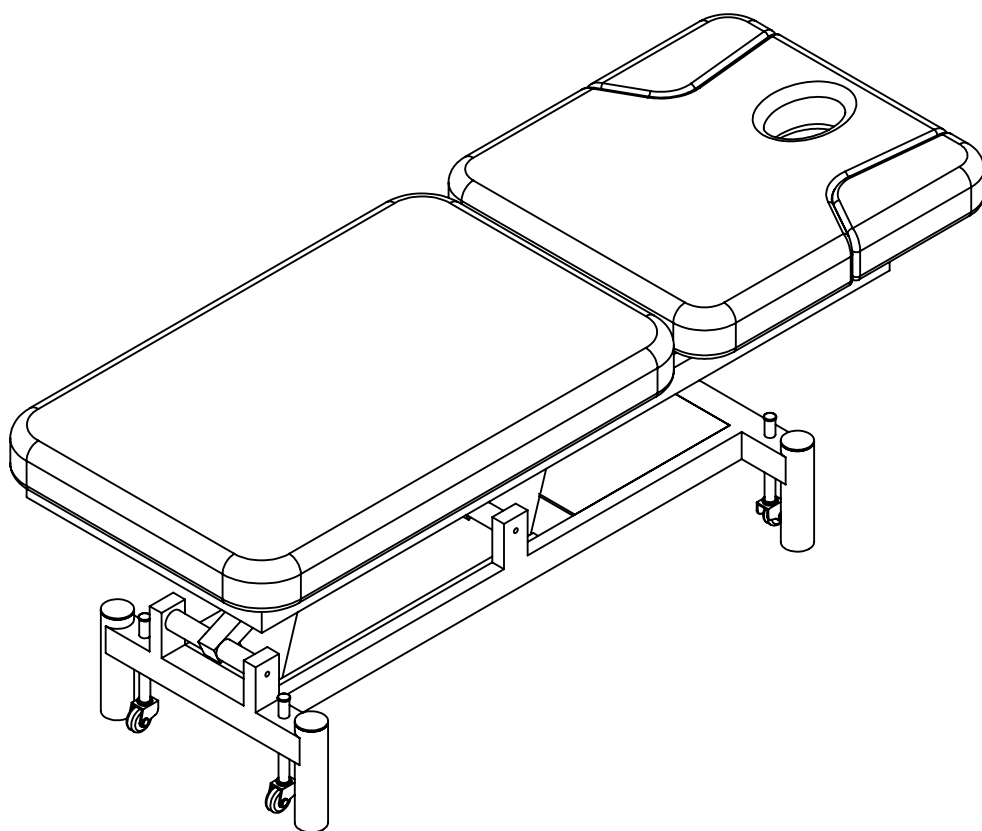



IMPORTANT PLEASE READ THESE SAFETY INSTRUCTIONS

Listed below are essential safety precautions.

GENERAL SAFETY

1. Read carefully and keep these instructions in a safe place.
2. Use this product only for its intended function and do not use attachments not recommended by the manufacturer.
3. Do not use outdoors.
4. Do not try to move the table when in use.

ELECTRICAL APPLIANCES

1. All Power Points/Sockets must be installed by a qualified Electrician in accordance with local electrical standards.
2. DO NOT allow children to operate the electric motors in this product.
3. Turn the power off and remove the plug when not in use.
4. Turn the power off and remove the plug when cleaning.
5. Do not let the power cord get damaged in any way.
6. Keep the power cord away from heat sources.
7. Do not use an extension cord unless the cord complies with local electrical safety standards.
8. Periodically inspect the power cord for signs of damage.
9. Do not operate any electrical item with a damaged power cord.



IMPORTANT

KEEP FINGERS AND HANDS AWAY FROM TABLE JOINTS UNDER THE TABLE



IMPORTANT

DO NOT ALLOW CHILDREN AROUND THE TREATMENT TABLE WHEN IN USE



IMPORTANT

DO NOT ALLOW CHILDREN TO OPERATE THE ELECTRIC MOTORS THAT MOVE THE TABLE



IMPORTANT




DO NOT USE AROUND WATER OR IN A WET ENVIRONMENT




IMPORTANT

WHEN REPLACING ANY PARTS OR ASSEMBLING THE TABLE, YOU MUST ENSURE THE TABLE IS RESTING ON ITS SIDE TO SAFEGUARD FROM CRUSH INJURY

GET STARTED

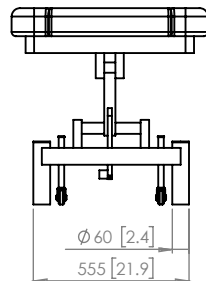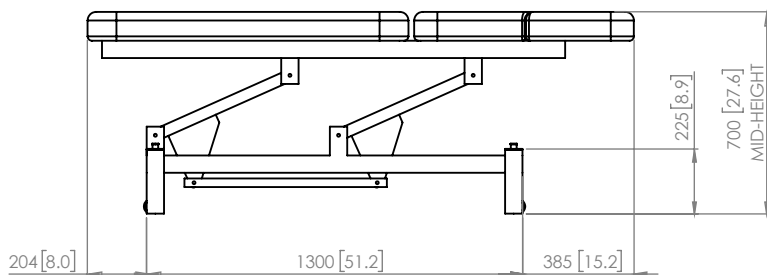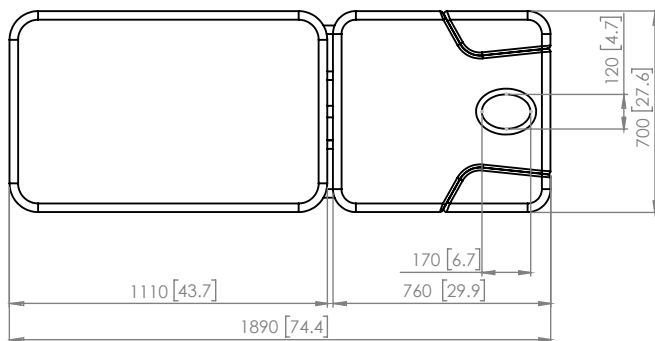

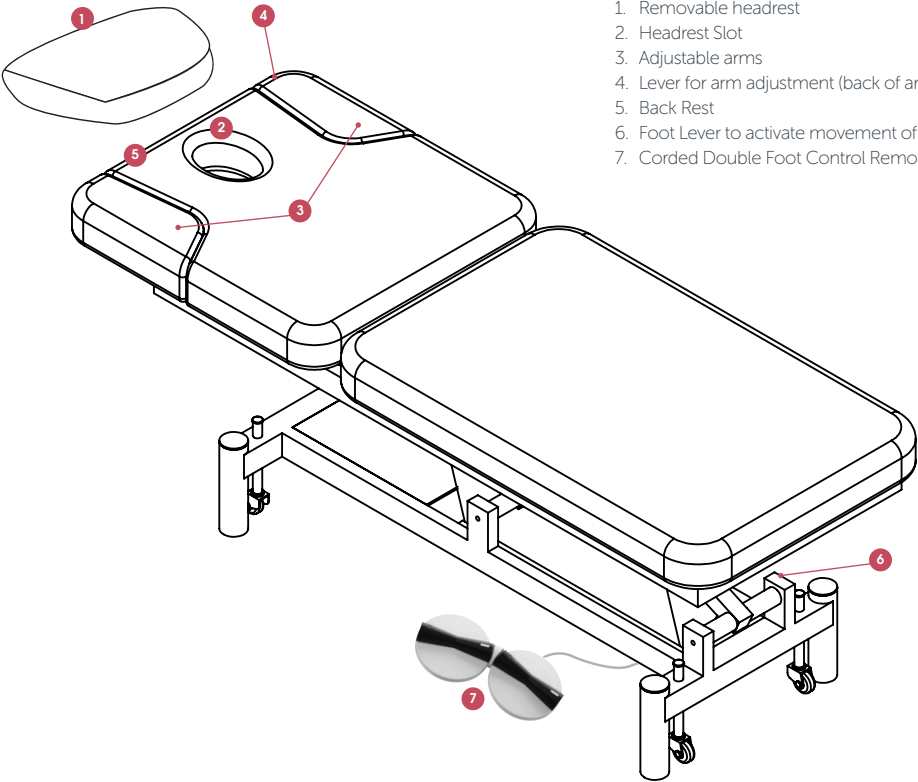

PARTS

- 1. Removable headrest
- 2. Headrest Slot
- 3. Adjustable arms
- 4. Lever for arm adjustment (back of arm)
- 5. Back Rest
- 6. Foot Lever to activate movement of wheels
- 7. Corded Double Foot Control Remote

INSTALLATION & ASSEMBLY: BATTERY BACK UP

For a Step by step guide to install the Back Up Battery on your Vista Treatment Table, refer to our Help & Support Website and watch the video:

Watch Video Here: <https://help.comfortel.com.au/solutions-troubleshooting/install-a-back-up-battery-on-a-treatment-table/>

How to Install a Battery Back Up



Back up power supply available for our treatment tables. A step by step guide on how to install a battery on a Gemini or Venus Treatment Table.

Watch Video



OPTIONAL UPGRADE BATTERY
for back up power supply #6090




Safety Certification:
Linak Medically
approved Electric Motors



Adjust the height and back rest range of the treatment table with the double electric foot control.

- A. FOR HEIGHT ADJUSTMENT (- MOVES BED DOWN + MOVES BED UP)
- B. FOR BACK REST ADJUSTMENT (- MOVES BACK REST BACK + MOVES BACK REST FORWARD)

FUNCTION

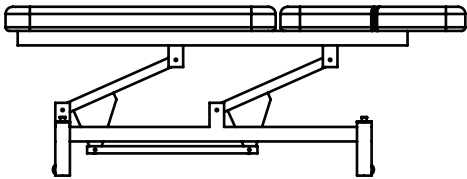


455- 910 MM

HEIGHT ADJUSTMENT


Using the Height adjustment foot control, adjust the table height by going up or down.

Minimum Height: 455mm
Maximum height: 910mm

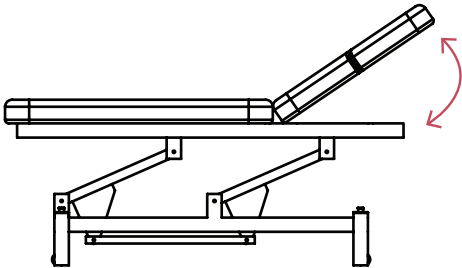


BACK ADJUSTMENT

Using the Back adjustment foot control, move back rest forward and back to flat position

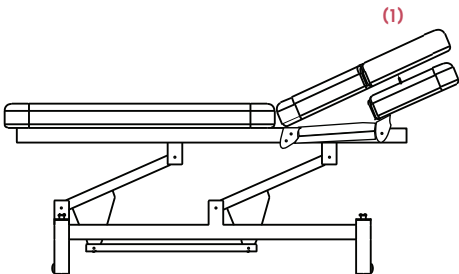

ARM ADJUSTMENT

Use the lever behind the arms to lower arms behind back rest. (1) with back-rest on an angle and (2) with back-rest flat down

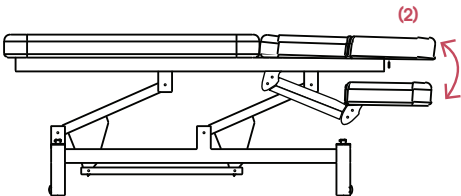

UPHOLSTERY

All upholstery needs some care; good housekeeping will help to prolong the life of any upholstered products. Avoid extreme temperatures. Extreme cold can make your vinyl prone to cracking, while extreme heat can cause permanent stretching. Avoid exposing your products to direct sunlight for prolonged periods of time.

ALWAYS FOLLOW THE CLEANING INSTRUCTIONS


- Wipe with a clean colourfast cloth dampened with mild alkaline free soap and luke warm water. Do not wet excessively.
- Remove all traces of any cleaning product by wiping with fresh water and a clean cloth.
- Dry with a soft, clean cloth.
- Stubborn grime may require gentle scrubbing with a soft brush.

Do not use:

- hot or boiling water
- hydrocarbons including alcohol
- acetone
- furniture polishes
- bleach
- concentrated detergents
- abrasive powder or paste cleaners
- steel wool
- silicon or petroleum-based cleaners
- solvent based or abrasive cleaning agents

MAINTENANCE

A periodic inspection of all components and fasteners should be made to ensure that your Comfortel Product is structurally intact and functioning properly.
Damaged and broken parts should be replaced and loose fasteners tightened.
All maintenance to be carried out by a suitably qualified tradesperson.

 **IMPORTANT**
WHEN REPLACING ANY PARTS OR ASSEMBLING THE TABLE, YOU MUST ENSURE THE TABLE IS RESTING ON IT'S SIDE TO SAFEGUARD FROM CRUSH INJURY

comfortel

help.comfortel.com.au

PRODUCT CODE

#6012.01 VISTA TREATMENT TABLE
1 X 10AMP GPO REQUIRED
POWER: 240V, 50-60HZ, MAX 2.4A
MAX LOAD: 200KG (440lbs)

©Comfortel Furniture. All rights are reserved. No part of this document may be photocopied, reproduced or translated to another language without written consent of Comfortel Furniture Pty Ltd.

The information contained in this document is subject to change without notice and should not be construed as a commitment by Comfortel Furniture Pty Ltd.

Comfortel Furniture Pty Ltd. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.


Comfortel Furniture Pty Ltd. shall not be liable for incidental or consequential damages in connection with or arising out of the use of this document and the program material.

PLEASE RETAIN THIS INFORMATION FOR FUTURE REFERENCE

WARRANTY PERIOD

TERMS AND CONDITIONS: For more information about our warranty, please refer to our website:

WARRANTY



PURCHASE DATE

INVOICE NUMBER

NOTE YOUR PURCHASE DATE & INVOICE NUMBER FOR FUTURE REFERENCE