

treatment tables battery backup

#6090

BATTERY BACK UP POWER SUPPLY FOR TREATMENT TABLES






Disconnect the mains cable to application at the power outlet

Remove the power cable from the control box by inserting a screwdriver into the locking clip marked.




Release the control box off the application by pressing the tab on the mounting clip

Open the lid to the control box by releasing the locking clips

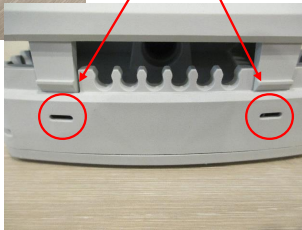



If you disconnect any actuator cables or handset cables take note of the correct ports

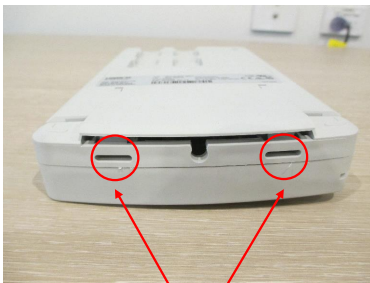

Open the lid to the battery box by releasing the locking clips


Open the lid for access

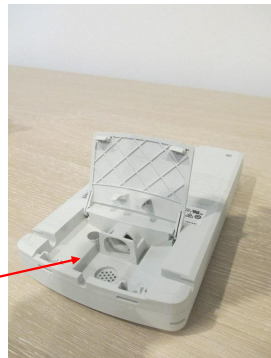

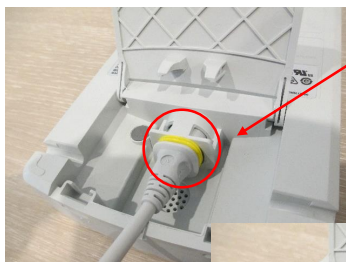


Insert the battery connection cable supplied in the battery port


Ensure it is fully connected

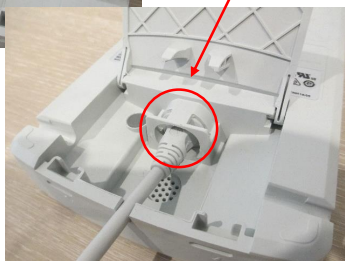


Close the lid ensuring the locking clips engage fully securing the lid

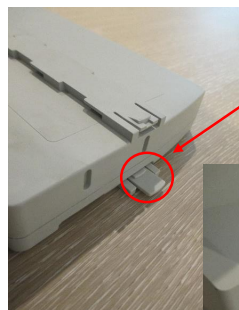

Connect the battery to the application ensuring the locking clip is fully engaged

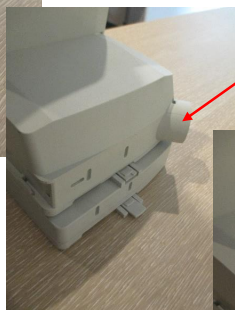

Connect the control box to the battery ensuring the locking clip is fully engaged

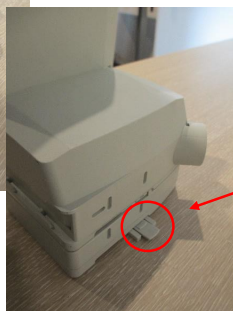


If the clips are not engaged fully the tab will be sticking out as indicated




Insert the battery connection cable into the battery port in the control box

Ensure it is fully connected

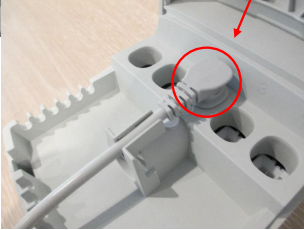

Reconnect any actuator cables or handset cables to the correct ports

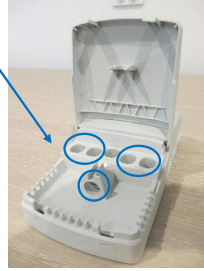

Close the control box lid ensuring the locking clips fully engage



Reconnect the mains cable to the control box ensure the locking clip engages

Turn on or reconnect the mains outlet

comfortel

help.comfortel.com.au

PRODUCT CODE

#6090 BATTERY BACKUP

Revision Date: 02.04.2019

©Comfortel Furniture 1997-2019 All rights are reserved.
No part of this document may be photocopied,
reproduced or translated to another language without
written consent of Comfortel Furniture Pty Ltd.

The information contained in this document is subject to change without notice and should not be construed as a commitment by Comfortel Furniture Pty Ltd.

Comfortel Furniture Pty Ltd. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Comfortel Furniture Pty Ltd. shall not be liable for incidental or consequential damages in connection with or arising out of the use of this document and the program material.

PLEASE RETAIN THIS INFORMATION FOR FUTURE REFERENCE

For more information about our products and services, contact us at comfortel.com.au