

AquaGlo

#7762

AQUAGLO





IMPORTANT PLEASE READ THESE SAFETY INSTRUCTIONS

GENERAL SAFETY & OPERATION

1. Read carefully and keep these instructions in a safe place.
2. Use this product only for its intended function and do not use attachments not recommended by the manufacturer.
3. Do not use outdoors.
4. Do not store or use the unit near water or at extreme temperatures, either cold or hot.
5. Do not expose unit to corrosive air or dust.
6. Do not expose the unit to flammables or any closed space where oxygen is used.
7. Use only approved solutions and cleaners for servicing unit.
8. Do not open, alter, renovate or disassemble the unit in any way except when instructed by Customer Service.
9. Stop the unit immediately if there is any error in operations.
10. Only trained, authorized person(s) should operate the unit.
11. To avoid excessive heat build up, unit should be powered down for 10-15 minutes after each hour of continuous use
12. Exposure to electrical surge or shock can damage the internal components of the unit. Ensure the unit's electrical cord is in good condition and is plugged into a surge protected outlet. Unplug the power cable if unit is not in use for a long period of time
13. Unit should be checked regularly and any damaged, broken part of the unit should be marked and the whole unit should be returned to distributor for service

BIOLOGICAL HAZARDS & CLIENT SAFETY

1. Residues such as blood, sebum, and skin debris in the suction bottle should be completely removed and cleaned after use.
2. All used tips must be immediately disposed of after treatment OR the correct sterilisation procedure (outlined in cleaning procedures) must be followed before the tips may be re-used. Failure to comply with this may result in transmission of infection
3. Extremely high suction pressure may cause damage to the client's skin. Select proper suction pressure according to skin conditions.

CONTRAINDICATIONS & PRECAUTIONS

CONTRAINDICATIONS

As with any beauty treatment, there are some individuals who may not be suited to receiving treatment with this equipment. We recommend that you **DO NOT** treat the following:

- Undiagnosed skin lesions
- Areas of damaged skin
- Active rosacea
- Large areas of inflammatory acne
- Clients taking RoAccutane
- Clients with a cold sore
- Eyelid areas
- Areas of eczema/dermatitis
- Very vascular (red) skins with fragile capillaries
- Clients with very thin or loose skin
- Clients taking medicines that cause thinning of skin (oral cortisone etc.)





PRECAUTIONS

- Cold sore sufferers should be aware that treating over the prone area may stimulate an outbreak of a cold sore (avoid area if possible)
- Clients who take blood thinning medications may be more prone to bruising or capillary damage. Treat with caution and use low vacuum settings.

AQUAGLO UNIT

COMPONENTS

1. Main housing
2. Solution 3 bottle
3. ON/OFF switch for solution 3
4. Solution 1 bottle
5. ON/OFF switch for solution 1
6. Storage tray
7. Hand piece
8. Suction 'IN' tube
9. Suction 'OUT' tube
10. ON/OFF switch
11. Suction bottle
12. Power cord
13. Vacuum control (right panel)
14. Flow control (left panel)

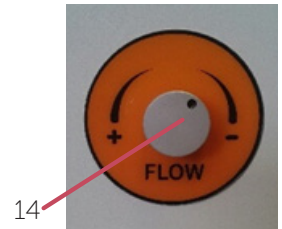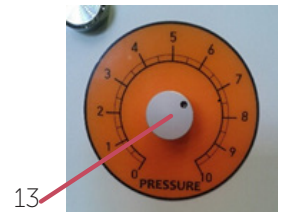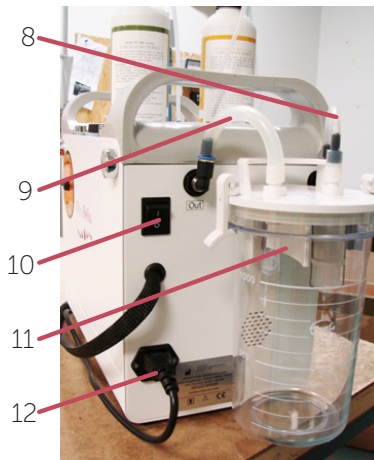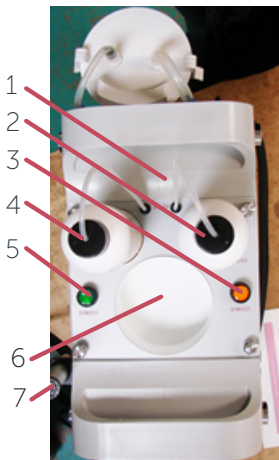

IN THE BOX

- | | |
|------------------------------------|-----|
| 1. Main unit | x1 |
| 2. Tips - large (disposable) | x10 |
| 3. Tips - small (disposable) | x10 |
| 4. Blue cleaning tips (disposable) | x2 |
| 5. White cleaning bottle | x1 |
| 6. Suction bottle | x1 |
| 7. Solution 1 bottle | x4 |
| 8. Solution 3 bottle | x2 |
| 9. Owners manual | x1 |
| 10. Power cable | x1 |

PREPARATION

1. Check the mains power supply matches the specifications (220V)
2. Check if switches, tubes and accessories are properly connected and if the unit works normally.
3. Refill the serum bottles if necessary
4. Check the condition and contents of suction bottles and tubes before use. Contents of the suction bottle should be cleaned out after every use. Wash with warm water and mild detergent, rinse clean before re-attaching to system.
5. Connect the suction bottle as shown. Make sure the bottle is properly installed on the unit and suction bottle cover is sealed correctly. Make sure the bottle has sufficient space to collect waste.
6. Ensure the stainless steel springs are firmly connected to the two large tubes.

Note:

Ensure only authorized, qualified persons operate the unit.

Read owners manual thoroughly before first use.



OPERATION

1. Install power cord into base and plug into power source and turn on main power switch
2. Press the on/off button for Solution 1 (green) or Solution 3 (orange) to start
3. Set vacuum setting to 3, and flow rate setting just above minimum to begin, then adjust according to client skin type and comfort, and treatment objective.
4. Put hand piece on skin and apply treatment
5. After finishing each session, ensure equipment is cleaned as per recommended procedure

CLEANING

1. Remove clear treatment tip from hand piece and fit the blue cleaning tip in its place.
2. Remove Solution 1 bottle and replace with white cleaning bottle filled with a 10:1 distilled water and white vinegar solution. Place suction tube and cap onto white cleaning bottle
3. Turn the flow dial to maximum.
4. Press the Solution 1 on/off button and run cleaning tip for 2 minutes.
5. after 2 minutes, repeat the same procedure with the solution 3 side of the system.
6. Turn off machine, empty cleaning bottle and rinse thoroughly.
7. Repeat steps 2 - 6 with distilled water only (no vinegar) in the cleaning bottles to flush the system of cleaning fluid.

STERILIZATION OF TREATMENT TIPS

All used tips must be immediately disposed of after treatment OR the correct sterilization procedure (outlined below) must be followed before the tips may be re-used. Washing with soap and water only is NOT an adequate method of cleaning the tips, and may result in the transmission of infection.

STERILIZATION PROCEDURE:

1. Wearing gloves, remove the used tips from the handpiece immediately after treatment
2. Using warm running water, mild alkaline detergent and a small bristle brush (a toothbrush is ideal), scrub any product residue and skin debris out of the tips and rinse thoroughly. Ensure all surfaces are scrubbed - inside and out.
3. Dry with lint-free cloth or paper towel
4. Prepare an industry-approved chemical sterilizing agent according to manufacturer instructions. Carefully follow out correct dilution instructions
5. soak the cleaned tips in the solution for the recommended amount of time (follow manufacturer instructions, as this will vary depending on chemical used and dilution recommended)
6. remove the tips from the solution, allow to air dry on paper towel
7. store the dry tips in a covered container until next use.

SPECIFICATIONS

Size: 350 (W) x 200 (D) x 323 (H)

Weight: 13kg

Vacuum: 550mmHg

Voltage: AC220V

Frequency: 50/60 Hz

Power: 100W

Ranges of vacuum suction adjustment: 0 - 10

Ranges of solution flow rate adjustment: LOW to HIGH

Noise level: Less than 60dB

Operation mode: Continuous

Operation and storage temperatures:

Operation temperatures

- Temperature: 15~40 C

- Humidity: 30~75%

Transport and storage temperatures

- Temperature: -15~50 C

- Humidity: 10~95%

WARRANTY PERIOD

PERIOD AND DESCRIPTION	PRODUCTS
12 MONTHS PARTS & LABOR	All products
2 YEARS PARTS ONLY	All products excluding small electrical appliances
3 YEARS PARTS ONLY	All Linak electric actuators
5 YEARS PARTS ONLY	Taps cartridge, Ceramic basins

TERMS AND CONDITIONS

For more information about our terms and conditions including warranty, please refer to our website.

CLEANING

GENERAL CLEANING

- We recommend the use of soapy water or approved cleaners.
- This product should not be cleaned with abrasive materials.
- Damage caused by any improper treatment is not covered by the product warranty

MAINTENANCE

A periodic inspection of all components and fasteners should be made to ensure that your Comfortel product is structurally intact and functioning properly.

Damaged and broken parts should be replaced and loose fasteners tightened.

All maintenance to be carried out by a suitably qualified tradesperson.

comfortel

comfortel.com.au

PRODUCT CODE

#7762 AQUAGLO

Revision Date: 11.06.2019

©Comfortel Furniture 1997-2019. All rights are reserved.
No part of this document may be photocopied, reproduced or translated to another language without written consent of Comfortel Furniture Pty Ltd.

The information contained in this document is subject to change without notice and should not be construed as a commitment by Comfortel Furniture Pty Ltd.

Comfortel Furniture Pty Ltd. assumes no responsibility for any errors that may appear in this document nor does it make expressed or implied warranty of any kind with regard to this material, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Comfortel Furniture Pty Ltd. shall not be liable for incidental or consequential damages in connection with or arising out of the use of this document and the program material.

PLEASE RETAIN THIS INFORMATION FOR FUTURE REFERENCE

For more information about our products and services, contact us at comfortel.com.au

PURCHASE DATE

INVOICE NUMBER

NOTE YOUR PURCHASE DATE & INVOICE NUMBER FOR FUTURE REFERENCE